


# Polymer Ligation System

## Click'aV<sup>®</sup> Ligating Clips & Clip Appliers

Safe and minimally invasive solution even for the most challenging cases

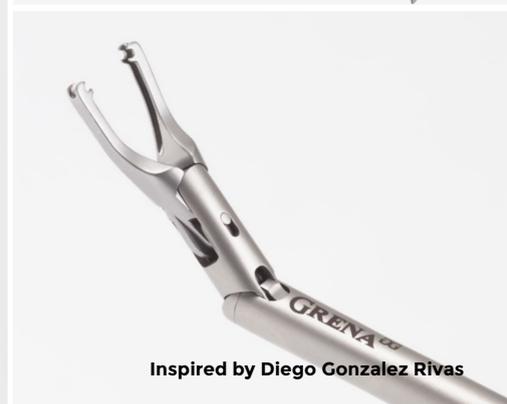

Inspired by Diego Gonzalez Rivas

PROVIDING SPECIALISTS WITH  
**INNOVATIVE**  
HEALTHCARE SOLUTIONS

WITH THE CENTRAL FOCUS ON  
PATIENTS' SAFETY AND USERS' COMFORT



## Click'aV Plus™ Ligating Clips

The second generation Click'aV Plus™ is a unique ligating clip equipped with **patented teeth** that significantly increase the patient safety thanks to:

- **Bi-directional tissue stabilization**
- **Increased transverse stability** – eliminates tendency to slide off the vessel due to blood pressure or manipulation in the operation field
- **Atraumatic stabilization of vessels**




In a number of procedures, Click'aV Plus™ ligating clips successfully replace endostaplers or the harmonic scalpel, combining highest security level with cost savings. Equally important is the low interference with CT scanning, radiolucency in X-rays and the great safety in MRIs resulting in the lack of artifacts. Every clips is fitted with the hinge and the latch tested in thousands of procedures.

**Click'aV Plus™ clips were specifically designed for:**


- large vessels with increased pressure
- locations with difficult access or reduced visibility
- locations where manipulation in the operation field results in higher risk of mechanical clips displacement



Transverse stability is a major factor protecting against clip slipping off. Our main objective during the design work on Click'aV Plus™ was to protect the patient from the possibility of clip displacement off the vessel. Result of Grena's Team efforts is the second generation polymer clip - the first clip in the world with ABS built-in.

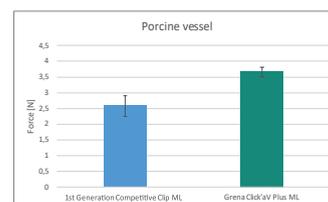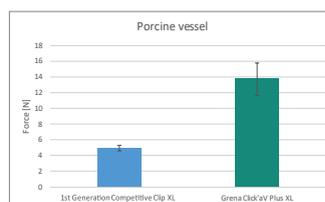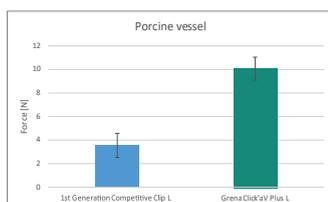

Click'aV<sup>Plus</sup>

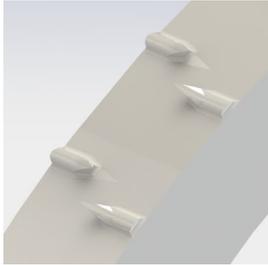

Patent No. 10,265,079

**KEY FEATURES:**

- 2, 3, 4 or 6 clips per cartridge
- 4 sizes available: ML, L, XL, XXL
- Atraumatic 2-ways stabilization
- Color-coded cartridges for easy identification
- Compatible with corresponding Click'aV<sup>®</sup> applicators
- Adhesive backing of the cartridge



REF	CLIP SIZE	TISSUE BUNDLE SIZE RANGE (MM)	COLOUR	PCS (per cartridge/ cartridges per box)
0301-10ML	ML	3-10	●	6/20
0301-10ML04	ML	3-10	●	4/20
0301-10ML02	ML	3-10	●	2/20
0301-10L	L	5-13	●	6/20
0301-10L04	L	5-13	●	4/20
0301-10L02	L	5-13	●	2/20
0301-10XL	XL	7-16	●	6/20
0301-10XL04	XL	7-16	●	4/20
0301-10XL02	XL	7-16	●	2/20
0301-10XXL04	XXL	10-22	●	4/8
0301-10XXL03	XXL	10-22	●	3/8
0301-10XXL02	XXL	10-22	●	2/8

**NEW**

**XXL Click'aV Plus<sup>™</sup> Ligating Clips**

With Click'aV Plus<sup>™</sup> XXL size surgeon can extend the range of economical benefits in procedures performed on extremely thick tissues by replacing expensive endostaplers or harmonic scalpel with the same safe, but cost effective technique.




**Unique size**  
**New possibilities**

## Click'aV<sup>®</sup> Ligating Clips

Click'aV<sup>®</sup> ligating clips are implantable medical devices made of an advanced polymer to ensure a high level of biocompatibility along with the structural and dimensional stability over time, necessary for long-lasting patient safety. They are designed to combine the speed of the metal clip with outstanding security comparable to 2.0 suture. This combination leads to extreme time savings.

Click'aV<sup>®</sup>

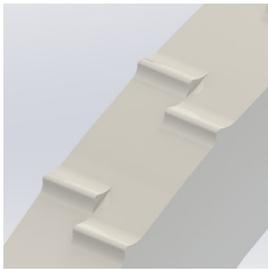

### KEY FEATURES:

- 2, 4, or 6 clips per cartridge
- 4 sizes available: M, ML, L, XL
- Color-coded cartridges for easy identification
- Compatible with corresponding Click'aV<sup>®</sup> Applicators
- Adhesive backing of the cartridge



REF	CLIP SIZE	TISSUE BUNDLE SIZE RANGE (MM)	COLOUR	PCS (per cartridge/ cartridges per box)
0301-03M	M	2-7	●	6/20
0301-03M04	M	2-7	●	4/20
0301-03ML	ML	3-10	●	6/20
0301-03ML04	ML	3-10	●	4/20
0301-03ML02	ML	3-10	●	2/20
0301-03L	L	5-13	●	6/20
0301-03L04	L	5-13	●	4/20
0301-03L02	L	5-13	●	2/20
0301-03XL	XL	7-16	●	6/20
0301-03XL04	XL	7-16	●	4/20
0301-03XL02	XL	7-16	●	2/20

## Click'aV<sup>®</sup> Open Surgery Clip Appliers

Click'aV<sup>®</sup> open surgery appliers are available in a wide range of sizes and applier lengths. Made of the highest quality surgical stainless steel to meet the requirements of the most demanding professionals. Handle rings are color-coded matching cartridge hues.

### KEY FEATURES:

- Available for M, ML, L and XL Click'aV<sup>®</sup> & Click'aV Plus<sup>™</sup> clips
- 2 lengths available: 20 cm and 27 cm
- Made of high quality surgical stainless steel



REF	CLIP SIZE	TIP STYLE	APPLIER LENGTH (CM)	COLOUR	PCS PER BOX
0301-04M20	M	Curved	20	●	1
0301-04M27	M	Curved	27	●	1
0301-04M27A70	M	Right Angled 70°	27	●	1
0301-04ML20	ML	Curved	20	●	1
0301-04ML27	ML	Curved	27	●	1
0301-04ML27A70	ML	Right Angled 70°	27	●	1
0301-04L20	L	Curved	20	●	1
0301-04L27	L	Curved	27	●	1
0301-04L27A70	L	Right Angled 70°	27	●	1
0301-04XL20	XL	Curved	20	●	1
0301-04XL27	XL	Curved	27	●	1
0301-04XL27A70	XL	Right Angled 70°	27	●	1

## Click'aV<sup>®</sup> Endoscopic Clip Appliers

Click'aV<sup>®</sup> endoscopic clip appliers are available in a wide range of sizes. Made of the highest quality surgical stainless steel Grena appliers guarantees ergonomic and effective approach in endoscopic procedures. Handle rings are color-coded matching cartridge hues. Precise performance makes Grena appliers the leading choice for surgeons performing endoscopic procedures.

### KEY FEATURES:

- Jaws activated by front handle for better stability
- 360° shaft rotation by index finger allows one-hand operation
- Flushing port facilitates easy cleaning, thus eliminating potential contamination
- Made of high quality surgical stainless steel
- Non-detachable design for easier maintenance and durability



REF	CLIP SIZE	TYPE	WORKING LENGTH (CM)	SHAFT/TRO-CAR DIAMETER (MM)	COLOUR	PCS PER BOX
0301-04ME	M	Endoscopic	33	5/5	●	1
0301-04MLE	ML	Endoscopic	33	5/5	●	1
0301-04LE	L	Endoscopic	33	10/10	●	1
0301-04XLE	XL	Endoscopic	33	10/10	●	1

## Click'aV<sup>®</sup> Endoscopic Clip Appliers 45 cm

In order to carry out bariatric procedures safely, Grena offers extra length endoscopic clip appliers for bariatric surgery.

### KEY FEATURES:

- Jaws activated by front handle for better stability
- 360° shaft rotation by index finger allows one-hand operation
- Flushing port facilitates easy cleaning, thus eliminating potential contamination
- Made of high quality surgical stainless steel
- Non-detachable design for easier maintenance and durability



REF	CLIP SIZE	TYPE	WORKING LENGTH (CM)	SHAFT/TRO-CAR DIAMETER (MM)	COLOUR	PCS PER BOX
0301-04MEB	M	Endoscopic	45	5/5	●	1
0301-04MLEB	ML	Endoscopic	45	5/5	●	1
0301-04LEB	L	Endoscopic	45	10/10	●	1
0301-04XLEB	XL	Endoscopic	45	10/10	●	1

## Click'aV<sup>®</sup> Endoscopic Clip Appliers 20°

20° endoscopic clip appliers were specifically designed for better access in difficult anatomical conditions.

### KEY FEATURES:

- Jaws activated by front handle for better stability
- 360° shaft rotation by index finger allows one-hand operation
- Flushing port facilitates easy cleaning, thus eliminating potential contamination
- Made of high quality surgical stainless steel
- Non-detachable design for easier maintenance and durability



REF	CLIP SIZE	TYPE	WORKING LENGTH (CM)	SHAFT/TRO-CAR DIAMETER (MM)	COLOUR	PCS PER BOX
0301-04MEA20	M	Endoscopic	33	5/10		1
0301-04MLEA20	ML	Endoscopic	33	5/10		1
0301-04LEA20	L	Endoscopic	33	10/12		1
0301-04XLEA20	XL	Endoscopic	33	10/12		1

## Click'aV<sup>®</sup> Detachable Endoscopic Clip Appliers

This products line is offered for users with the highest hygienic regimes. The device can be disassembled to allow medical personnel to clean and disinfect it according to the applicable local requirements without any compromise. The other advantage of this development is very high flexibility and compatibility across the components.



REF	CLIP SIZE	PRODUCT	WORKING LENGTH (CM)	SHAFT/TRO-CAR DIAMETER (MM)	COLOUR	PCS PER BOX
0301-04MMLEHS	M/ML	Handle with shaft	33	5/5		1
0301-04LXLEHS	L/XL	Handle with shaft	33	10/10		1

REF	CLIP SIZE	PRODUCT	FITS	PCS PER BOX
0301-04MEI	M	Insert	0301-04MMLEHS	10
0301-04MLEI	ML	Insert	0301-04MMLEHS	10
0301-04LEI	L	Insert	0301-04LXLEHS	10
0301-04XLEI	XL	Insert	0301-04LXLEHS	10

## Click'aV<sup>®</sup> Universal Endoscopic Clip Appliers

Listening to the suggestions of surgeons we designed a universal clip applicator (Patent Pending) that can be used to load either L or XL polymer clips. With an easy switch of a lever the operator decides what clip size is to be used with the device. The concept helps to reduce the number of applicators held by the hospital as well as reduce the environmental footprint related to cleaning, disinfection and sterilization of applicators.



REF	CLIP SIZE	TYPE	WORKING LENGTH (CM)	SHAFT/TROCAR DIAMETER (MM)	COLOUR	PCS PER BOX
0301-04LXLUNE	L/ XL	Endoscopic	33	10/10		1

## Gonzalez Rivas Endoscopic Clip Appliers 45°

The 45° endoscopic clip applier is the unique device designed in collaboration with Dr. Diego Gonzalez Rivas to allow better access in VATS surgery. It facilitates clip application in the most challenging VATS cases, especially in single access port procedures.



**Dr. Diego Gonzalez Rivas**

Degree in Medicine and Surgery by the Universidad de Santiago de Compostela (1992-1998). Resident at the Thoracic Surgery Unit at Juan Canalejo hospital in La Coruna from 1999 to 2004. Medical doctor in Thoracic Surgery since July 2004, with medical practice at the Thoracic Surgery Unit at Complejo Hospitalario Universitario in Santiago de Compostela until April 2005 and from then in the Thoracic Surgery Unit at Complejo Hospitalario Universitario de A Coruna.



REF	CLIP SIZE	TYPE	WORKING LENGTH (CM)	SHAFT/ TROCAR DIAMETER (MM)	RETRACTOR (recommended)	COLOUR	PCS PER BOX
0301-04MEA45	M	Endoscopic	33	5/10	RingProtect™ Gonzalez Rivas modified	●	1
0301-04MLEA45	ML	Endoscopic	33	5/10	RingProtect™ Gonzalez Rivas modified	●	1
0301-04LEA45	L	Endoscopic	33	10/NA	RingProtect™ Gonzalez Rivas modified	●	1
0301-04XLEA45	XL	Endoscopic	33	10/NA	RingProtect™ Gonzalez Rivas modified	●	1

## Omnifinger™ Articulating Endoscopic Clip Appliers

---


Omnifinger™ is a unique articulating endoscopic clip applier designed in collaboration with Dr. Diego Gonzales Rivas. It was specifically developed to deal with challenging cases.

### KEY FEATURES:

- 60° of total angulation
- 2 shaft diameter available: 5 mm and 10 mm
- 2 lengths available: 36 cm and 47 cm
- Articulation and 360° shaft rotation by index finger allows one-hand operation
- Flushing port facilitates easy cleaning, thus eliminating potential contamination
- Made of high quality surgical stainless steel
- By default without lock, version with lock available for special order



Patent pending

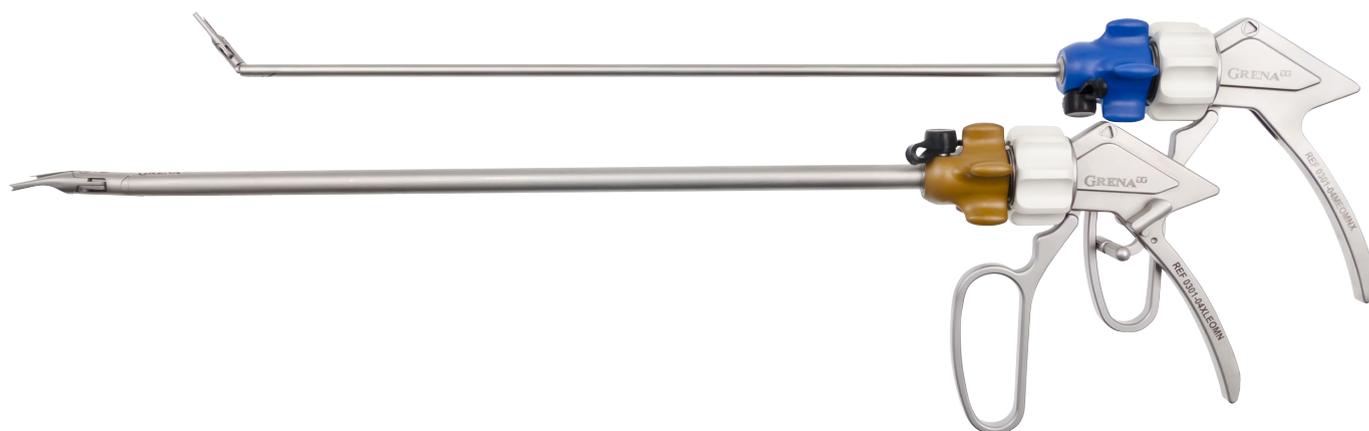

REF	SIZE	LOCK	COLOUR	SHAFT DIAMETER (MM)	TROCAR DIAMETER (MM)	WORKING LENGTH (CM)	BOX QUANTITY (PCS)
0301-04MEOMN	M	YES	●	5	5	36	1
0301-04MEOMNX	M	NO	●	5	5	36	1
0301-04MLEOMN	ML	YES	●	5	5	36	1
0301-04MLEOMNX	ML	NO	●	5	5	36	1
0301-04LEOMN	L	YES	●	10	10	36	1
0301-04LEOMNX	L	NO	●	10	10	36	1
0301-04XLEOMN	XL	YES	●	10	10	36	1
0301-04XLEOMNX	XL	NO	●	10	10	36	1
0301-04MEOMNB	M	YES	●	5	5	47	1
0301-04MEOMNBX	M	NO	●	5	5	47	1
0301-04MLEOMNB	ML	YES	●	5	5	47	1
0301-04MLEOMNBX	ML	NO	●	5	5	47	1
0301-04LEOMNB	L	YES	●	10	10	47	1
0301-04LEOMNBX	L	NO	●	10	10	47	1
0301-04XLEOMNB	XL	YES	●	10	10	47	1
0301-04XLEOMNBX	XL	NO	●	10	10	47	1

## Clip Removers

---



REF	CLIP SIZE	PRODUCT	TIPS	PCS PER BOX
0301-R804MMLL	M, ML, L	Open Surgery Clip Remover	-	1
0301-R804XL	XL	Open Surgery Clip Remover	-	1
0301-R804MLLE	M, ML, L	Endoscopic Clip Remover	-	1
0301-R804LXLE	L, XL	Endoscopic Clip Remover	Recommended for L	1
0301-R804MLLEB	M, ML, L	Endoscopic Clip Remover Bariatric	-	1
0301-R804LXLEB	L, XL	Endoscopic Clip Remover Bariatric	Recommended for L	1

# INDEX

Extensive list of the available clip appliers and clip removers

REF	PRODUCT
0301-04L20	Click'aV® Ligating Clip Applier L, 20cm, open surg
0301-04L27	Click'aV® Ligating Clip Applier L, 27cm, open surg
0301-04L27A70	Click'aV® Ligating Clip Applier L, 27cm, 70°, open surg
0301-04LE	Click'aV® Ligating Clip Applier L, endo
0301-04LEA20	Click'aV® Ligating Clip Applier L, endo, angled 20°
0301-04LEA20B	Click'aV® Ligating Clip Applier L, endo, angled 20°, bariatric
0301-04LEA20I	Click'aV® Ligating Clip Applier Insert, L, endo, angled 20°
0301-04LEA20IB	Click'aV® Ligating Clip Applier Insert, L, endo, angled 20°, bariatric
0301-04LEA45	Click'aV® Ligating Clip Applier L, endo, Gonzalez Rivas
0301-04LEA45I	Click'aV® Ligating Clip Applier Insert, L, endo, Gonzalez Rivas
0301-04LEA45IB	Click'aV® Ligating Clip Applier Insert, L, endo, Gonzalez Rivas, bariatric
0301-04LEB	Click'aV® Ligating Clip Applier L, endo, 45 cm, bariatric
0301-04LEI	Click'aV® Ligating Clip Applier Insert, L, endo
0301-04LEIB	Click'aV® Ligating Clip Applier Insert, L, endo, bariatric
0301-04LEOMN	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, L, endo
0301-04LEOMNB	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, L, endo, bariatric
0301-04LEOMNBX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, L, endo, bariatric, without lock
0301-04LEOMNX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, L, endo, without lock
0301-04LXLEHS	Click'aV® Ligating Clip Applier Handle with Shaft, L, XL, endo
0301-04LXLEHSB	Click'aV® Ligating Clip Applier Handle with Shaft, L, XL, endo, 45 cm (bariatric)
0301-04M20	Click'aV® Ligating Clip Applier M, 20cm, open surg
0301-04M27	Click'aV® Ligating Clip Applier M, 27cm, open surg,
0301-04M27A70	Click'aV® Ligating Clip Applier M, 27cm, 70°, open surg,
0301-04ME	Click'aV® Ligating Clip Applier M, endo
0301-04MEA20	Click'aV® Ligating Clip Applier M, endo, angled 20°
0301-04MEA20B	Click'aV® Ligating Clip Applier M, endo, angled 20°, bariatric
0301-04MEA20I	Click'aV® Ligating Clip Applier Insert, M, endo, angled 20°
0301-04MEA20IB	Click'aV® Ligating Clip Applier Insert, M, endo, angled 20°, bariatric
0301-04MEA45	Click'aV® Ligating Clip Applier M, endo, Gonzalez Rivas
0301-04MEA45I	Click'aV® Ligating Clip Applier Insert, M, endo, Gonzalez Rivas
0301-04MEA45IB	Click'aV® Ligating Clip Applier Insert, M, endo, Gonzalez Rivas, bariatric

REF	PRODUCT
0301-04MEB	Click'aV® Ligating Clip Applier M, endo, bariatric
0301-04MEHS	Click'aV® Ligating Clip Applier Handle with Shaft, M endo
0301-04MEI	Click'aV® Ligating Clip Applier Insert, M, endo
0301-04MEIB	Click'aV® Ligating Clip Applier Insert, M, endo, bariatric
0301-04MEOMN	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, M, endo
0301-04MEOMNB	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, M, endo, bariatric
0301-04MEOMNBX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, M, endo, bariatric, without lock
0301-04MEOMNX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, M, endo, without lock
0301-04ML20	Click'aV® Ligating Clip Applier ML, 20cm, open surg
0301-04ML27	Click'aV® Ligating Clip Applier ML, 27cm, open surg
0301-04ML27A70	Click'aV® Ligating Clip Applier ML, 27cm, 70°, open surg
0301-04MLE	Click'aV® Ligating Clip Applier ML, endo
0301-04MLEA20	Click'aV® Ligating Clip Applier ML, endo, angled 20°
0301-04MLEA20B	Click'aV® Ligating Clip Applier ML, endo, angled 20°, bariatric
0301-04MLEA20I	Click'aV® Ligating Clip Applier Insert, ML, endo, angled 20°
0301-04MLEA20IB	Click'aV® Ligating Clip Applier Insert, ML, endo, angled 20°, bariatric
0301-04MLEA45	Click'aV® Ligating Clip Applier ML, endo, Gonzalez Rivas
0301-04MLEA45I	Click'aV® Ligating Clip Applier Insert, ML, endo, Gonzalez Rivas
0301-04MLEA45IB	Click'aV® Ligating Clip Applier Insert, ML, endo, Gonzalez Rivas, bariatric
0301-04MLEB	Click'aV® Ligating Clip Applier ML, endo, 45 cm, bariatric
0301-04MLEHS	Click'aV® Ligating Clip Applier Handle with Shaft, ML endo
0301-04MLEI	Click'aV® Ligating Clip Applier Insert, ML, endo
0301-04MLEIB	Click'aV® Ligating Clip Applier Insert, ML, endo, bariatric
0301-04MLEOMN	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, ML, endo
0301-04MLEOMNB	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, ML, endo, bariatric
0301-04MLEOMNBX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, ML, endo, bariatric, without lock
0301-04MLEOMNX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, ML, endo, without lock
0301-04MMLEHS	Click'aV® Ligating Clip Applier Handle with Shaft, M, ML, endo
0301-04MMLEHSB	Click'aV® Ligating Clip Applier Handle with Shaft, M, ML, endo, 45 cm (bariatric)
0301-04XL20	Click'aV® Ligating Clip Applier XL, 20cm, open surg
0301-04XL27	Click'aV® Ligating Clip Applier XL, 27cm, open surg

REF	PRODUCT
0301-04XL27A70	Click'aV® Ligating Clip Applier XL, 27cm, 70°, open surg
0301-04XLE	Click'aV® Ligating Clip Applier XL, endo
0301-04XLEA20	Click'aV® Ligating Clip Applier XL, endo, angled 20°
0301-04XLEA20B	Click'aV® Ligating Clip Applier XL, endo, angled 20°, bariatric
0301-04XLEA20I	Click'aV® Ligating Clip Applier Insert, XL, endo, angled 20°
0301-04XLEA20IB	Click'aV® Ligating Clip Applier Insert, XL, endo, angled 20°, bariatric
0301-04XLEA45	Click'aV® Ligating Clip Applier XL, endo, Gonzalez Rivas
0301-04XLEA45I	Click'aV® Ligating Clip Applier Insert, XL, endo, Gonzalez Rivas
0301-04XLEA45IB	Click'aV® Ligating Clip Applier Insert, XL, endo, Gonzalez Rivas, bariatric
0301-04XLEB	Click'aV Ligating Clip Applier XL, endo, 45 cm, bariatric
0301-04XLEHS	Click'aV Ligating Clip Applier Handle with Shaft, XL endo
0301-04XLEI	Click'aV® Ligating Clip Applier Insert, XL, endo
0301-04XLEIB	Click'aV® Ligating Clip Applier Insert, XL, endo, bariatric
0301-04XLEOMN	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, XL, endo
0301-04XLEOMNB	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, XL, endo, bariatric
0301-04XLEOMNBX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, XL, endo, bariatric, without lock
0301-04XLEOMNX	OMNIFinger™ Articulating Click'aV® Ligating Clip Applier, XL, endo, without lock
0301-04LXLUNE	Universal Click'aV® Ligating Clip Applier, L/XL, endo
0301-04MLEN	Click'aV® Ligating Clip Applier ML, endo
0301-04MEN	Click'aV® Ligating Clip Applier M, endo
0301-04MEA20N	Click'aV® Ligating Clip Applier M, endo, angled 20°
0301-04MEA45N	Click'aV® Ligating Clip Applier M, endo, angled 45°
0301-04MEA20BN	Click'aV® Ligating Clip Applier M, endo, angled 20°, bariatric
0301-04MEA45BN	Click'aV® Ligating Clip Applier M, endo, angled 45°, bariatric
0301-04MEBN	Click'aV® Ligating Clip Applier M, endo, bariatric
0301-04MLEA20N	Click'aV® Ligating Clip Applier ML, endo, angled 20°
0301-04MLEA45N	Click'aV® Ligating Clip Applier ML, endo, angled 45°
0301-04MLEBN	Click'aV® Ligating Clip Applier ML, endo, bariatric
0301-04MLEA20BN	Click'aV® Ligating Clip Applier ML, endo, angled 20°, bariatric
0301-04MLEA45BN	Click'aV® Ligating Clip Applier ML, endo, angled 45°, bariatric

REF	PRODUCT
0301-04LEN	Click'aV® Ligating Clip Applier L, endo
0301-04LEBN	Click'aV® Ligating Clip Applier L, endo, bariatric
0301-04XLEN	Click'aV® Ligating Clip Applier XL, endo
0301-04XLEBN	Click'aV® Ligating Clip Applier XL, endo, bariatric
0301-04XXLEN	Click'aV® Ligating Clip Applier XXL, endo
0301-04XXLEBN	Click'aV® Ligating Clip Applier XXL, endo, bariatric
0301-04XXL20	Click'aV® Ligating Clip Applier XXL, 20 cm, open surgery
0301-R804XLLE	Click'aV® Ligating Clip Remover, L, XL, endo
0301-R804XLLEB	Click'aV® Ligating Clip Remover, L, XL, endo, bariatric
0301-R804XLEHS	Click'aV® Ligating Clip Remover Handle with Shaft, L, XL, endo
0301-R804XLEHSB	Click'aV® Ligating Clip Remover Handle with Shaft, L, XL, endo, 45 cm (bariatric)
0301-R804XLEI	Click'aV® Ligating Clip Remover Insert, endo, L, XL,
0301-R804XLEIB	Click'aV® Ligating Clip Remover Insert, endo, L, XL, bariatric
0301-R804MLLE	Click'aV® Ligating Clip Remover, M, ML, L, endo
0301-R804MLLEB	Click'aV® Ligating Clip Remover, M, ML, L, endo, bariatric
0301-R804MLLEHS	Click'aV® Ligating Clip Remover Handle with Shaft, M, ML, L endo
0301-R804MLLEHSB	Click'aV® Ligating Clip Remover Handle with Shaft, M, ML, L endo, 45 cm (bariatric)
0301-R804MLLEI	Click'aV® Ligating Clip Remover Insert, endo, M, ML, L
0301-R804MLLEIB	Click'aV® Ligating Clip Remover Insert, endo, M, ML, L, bariatric
0301-R804MMLL	Click'aV® Ligating Clip Remover M, ML, L, open surgery
0301-R804XL	Click'aV® Ligating Clip Remover XL, open surgery



## Grena Ltd.

1000 Great West Road,  
Brentford, London TW8 9HH,  
United Kingdom

## CUSTOMER SERVICE CENTER

NOTTINGHAM OFFICE, UK

+ 44 115 9704 800

+ 44 115 9704 800

export@grena.co.uk

[www.grena.co.uk](http://www.grena.co.uk)



LONDON CHAMBER  
COMMERCE AND INDUSTRY