

**ISDUBB SPETS HÄRDAD / ICE PICK SVCR,
ISDUBB KRONAN / ICEPICK CROWN**

Artikelnummer/ 123300
Article number 123350

Avsedd användning: Isdubben fästes på kryckan för att ge bättre grepp vid isigt underlag. Isdubben passar på kryckor 17-23 mm.


Intended Use: The icepick is intended to use on crutches on slippery/icy surfaces. The icepick fits crutches 17-23 mm.

Säkerhetsanvisning:

- Isdubben skruvas fast på kryckan med medföljande skruvar (1).
- För att fälla upp/ner isdubben trycks de båda plastknopparna in för att frigöra dubben (2).
- Säkerställ att isdubben är fixerad i position innan användning (3).
- Kryckans doppsko skall kontrolleras regelbundet. Skadade eller utslitna doppskor ska bytas ut (4).
- Allvarliga händelser som inträffar under användande skall rapporteras till Swereco

Safety Instructions:

- The icepick is mounted on the crutch with the screws delivered with the product (1).
- Push the plastic knobs to release the icepick to fold it up and down on the crutch tip (2).
- Make sure the icepick is properly fixed in position before use (3).
- The crutch tip should be checked regularly. Damaged or worn crutch tips should be replaced (4).
- Serious accidents occurring through use shall be reported to Swereco.

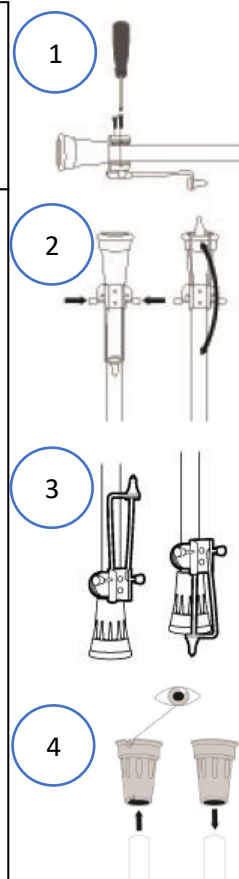

Teknisk specifikation / Technical specification

Art nr / art no: 123300

Isdubb spets hårdad/
Icepick SVCR

Art nr / art no: 123350

Icepick Kronan
Icepick Crown

Material: stainless steel, PP

Skötselansvisning: Rengörs regelbundet vid behov (min 1ggr/månad) med tvållösning, ph 5-9. Vid byte av brukare skall produkten desinficeras, autoklaveras i max 85° i max 3 minuter. Alternativt torkas med 70% isopropanol.

Care Instruction: Clean regularly if necessary (min once/month) with soap solution, ph 5-9. When changing users, the product should be disinfected, autoclaved at a maximum of 85° for a maximum of 3 minutes. Alternatively, dry with 70% isopropanol.

Återvinning: Då produkten är uttjänt återvinns den som metallsrot. Emballaget består av kartong och plast och återvinns i enlighet med detta.

Recycling: When the product is taken out of use, it is recycled as scrap metal. The packaging consists of cardboard and plastic and is recycled accordingly.



alkalisk tvållösning /
alkaline soap solution,
ph 5-9.



maximum of 85°C for a
maximum of 3 minutes



Visuell kontroll /
visual control



MDR 2017/745 2007/47/EC



Swereco AB
Vagnvägen 4
SE-364 42 Lenhovda
Tel: +46 (0)8 5000 5240
info@swereco.com
www.swereco.com