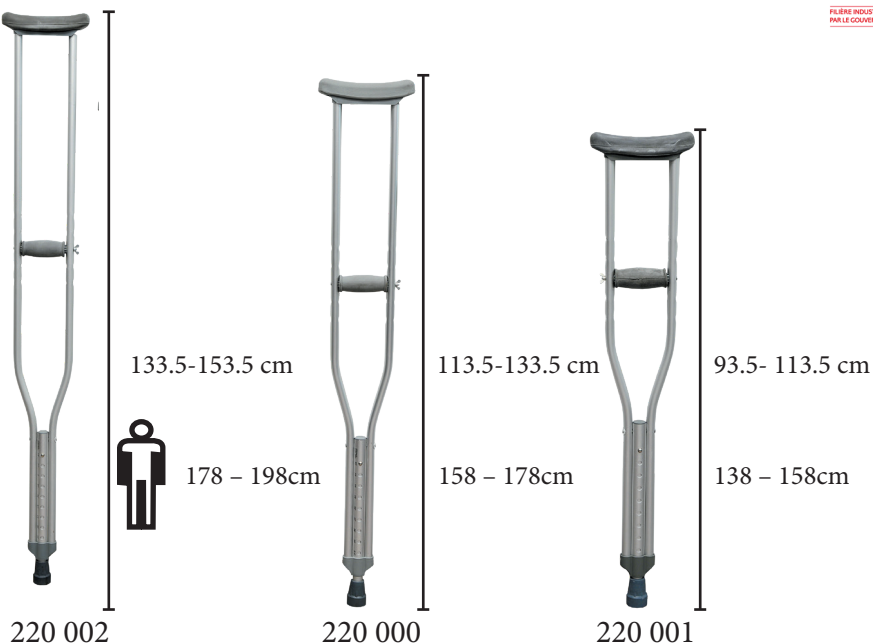
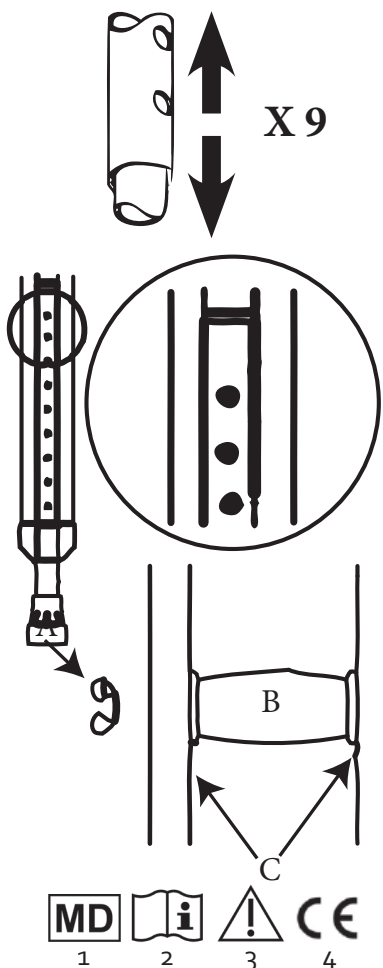
220000	0,9 kg	130 kg
220001	0,8 kg	130 kg
220002	1 kg	130 kg

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EN- Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all parts are tightened (handle, rubber tip..).
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling: let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces (snow, ice, water..)!)
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way: don't use as a lever or to move furniture.
- ♥ ALWAYS observe the weight limit on the labelling of your crutch.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ DO NOT hang anything on the Walking aid.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.
- ♥ Be careful when using: beware holes and deformations in the floor.
- ♥ The suction cap must not be too worn down. If the cap shows considerable wear, ask your supplier or nursing staff to replace it.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications

Intended use:

Walking aids are used for two purposes: as part of a rehabilitation programme when the user is recovering from an injury or operation and as a long-term aid to mobility when the user has a permanent difficulty with walking.

These devices help in maintaining balance, aid in the prevention of injuries (particularly falls), and allow the person to attend to his/her errands independently. Mobility aids can be used indoor or outdoor, depending on the model.

Mobility aids are used to modify the weight redistribution: some of the weight carried through the legs when walking is transferred through the arms of the frame or stick as it is leant on for support. Products equipped with a seat allow short rest to the user.

Indications

- Loss of autonomy due to walking disorders
- Balance and/or posture disorders
- Walking problems
- Rehabilitation programme after an injury or an operation

Contraindication

Don't use in case of:

- Severe cognitive disorder
- Severe loss of balance
- Severe motor or perception disorders
- Weakness in the upper limbs

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

Height Adjustment: grip stick shaft with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft. Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pin in front of the required hole position and tighten locking ring. The axillary crutches have 9 adjusting hole on the outside pipe, what means 9 adjusting positions. They can be adjusted every 2 cm.

Handle adjustments: the handle can be adjusted in 5 positions. Keep Hand Grip Wing Nut tightened.

Keep the axillary crutch close to your body while going to avoid accidents.

3. Cleaning

The axillary crutch has to be cleaned regularly with a mild cleaning agent and a soft cloth. NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the walking aid regularly.

The rubber tips must be checked and changed regularly, do not hesitate to ask your retailer.

4. Materials

Tube in anodised aluminium and polypropylene handle. Axillary support padded with SBS.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 24 months warranty, from the date of purchase, against any manufacturing defect.

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